PROSTHESIS SCAPHO-TRAPEZIO-TRAPEZOÏD

INCATM





Depuis 1714

SURGICAL TECHNIQUE

Approach




• Lateral approach centered on the scaphotrapezial joint space.

Scaphoid preparation



• Resection of the scaphoid distal pole (2mm).

• Remove the cut bone part.

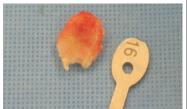

 Measure the size of this removed part using the gauge (MIAGT001) to determine the size of the implant required.

IMPORTANT

if the measurement of the resected bone part is between the 2 sizes of the gauge, you will have to choose the smaller size. Do not oversize the implant



• Using the shifted peak (MIAPC001), mark the entry point of the rasp for preparing the passage of the implant keel.




• Push in the rasp (MIARQ002) to create the location of the keel.

Implant placement



 Position the implant determinated by the initial measurement by placing the keel in the hole made in the scaphoid.

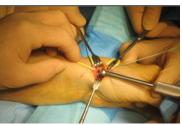

• Impact the implant using the impactor (MIAID001).



Closure and post-operative evolution



Close the capsular flap.

• Immobilize for 6 weeks.

Implant removal

An osteotome can be used for removing the implant if necessary.

IMPLANTS

INCA™ STT PROTHESIS

- cobalt-chromium-molybdenum alloy.
- cementless fixation: titanium + HA coating.
- anatomical bearing surface with mirror-polished finish.
- keel ensuring primary stability.
- sterilized by Gamma irradiation.

STT INCA™



DesignationINCA HA STT COCR 14/10
INCA HA STT COCR 16/11

Ref

MIIHC014 MIIHC016

INSTRUMENTATION

TO STERILIZE

DESIGNATION	REF
INCA STT GAUGE	MIAGT001
INCA STT IMPACTOR	MIAID001

INCA STT RASP

INCA STT SHIFTED PEAK

lépine Algérie

9 rue philosophe Tabrizi Les sources Bir Mourad Rais - ALGER lepine-algerie@groupe-lepine.com

lépine Ibérica

J.J. Tadeo Murguía, 3 y 5 Bajos 20304 IRÚN (GUIPÚZCOA) lepine-iberica@groupe-lepine.com

lépine Colombia

Carrera 49 B #93-49 Barrio la Castellana 110221 BOGOTA lepine-colombia@groupe-lepine.com

lépine Italia

Via Cassanese 100, 20090 SEGRATE (MILANO) lepine-italia@groupe-lepine.com

lépine Deutschland

Ohle Ring 23-25 STADE-WIEPENKATHEN, 21684 lepine-deutschland@groupe-lepine.com

lépine Maroc

79 avenue IBN SINA 10080 RABAT - AGDAL lepine-maroc@groupe-lepine.com 175 rue Jacquard - CS 50307 69727 Genay Cedex - FRANCE Tél. +33 (0)4 72 33 02 95 Fax +33 (0)4 72 35 96 50

www.groupe-lepine.com



MIAPC001

MIARQ001

MIARQ002